


Easy Implant Removal

EFR KIT

- Bone loss-less fixture removal
- Quick & Simple removal by two(2) Stage
- Above 80% removal success


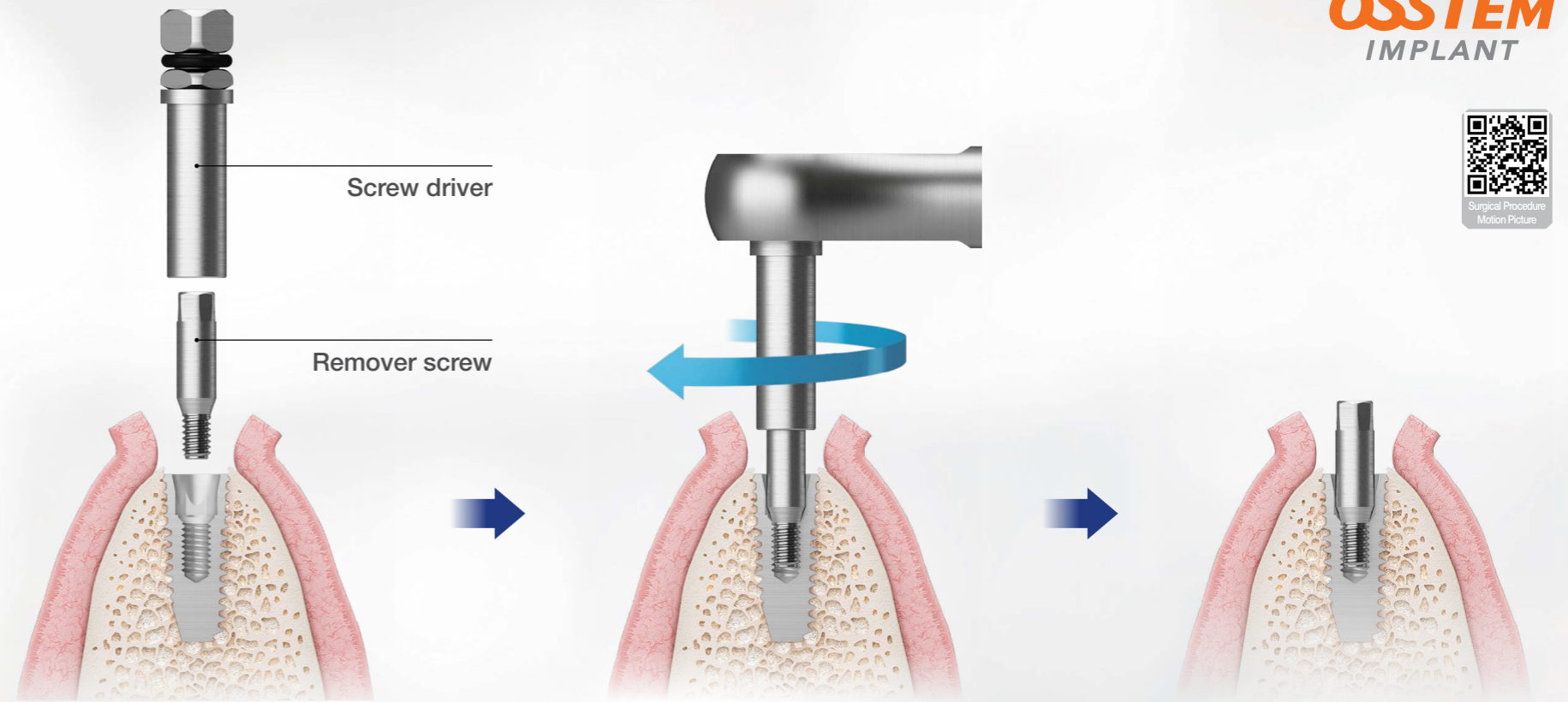


1. Select remover screw
(Refer to the remover screw selecting guide)

2. Clockwise tightening by torque wrench through the screw driver
(mini 80Ncm, regular 100Ncm)

3. Complete the removal screw installation into the fixture

Remover Screw Selection Guide

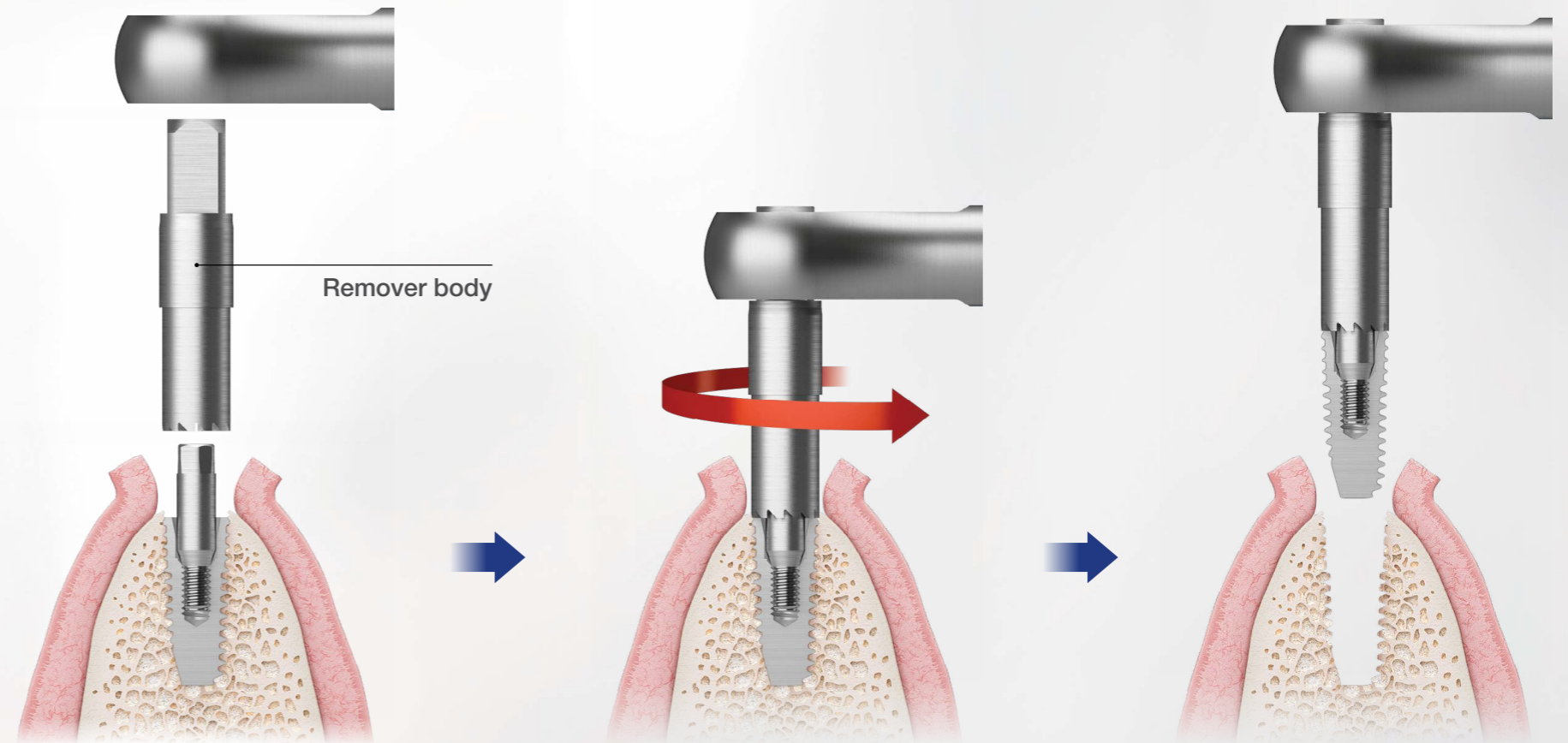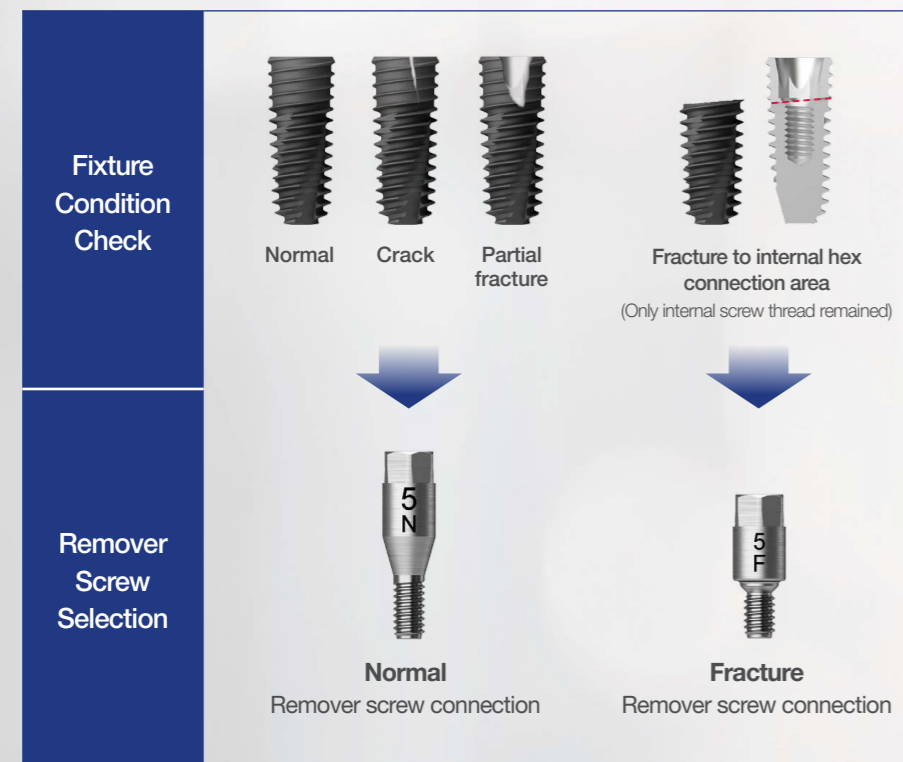

4. Removal body selection

5. Anti-clockwise screwing by torque wrench through the removal body
(Maximum 400Ncm engagement possible)

6. Bone loss-less fixture removal by strong adhesion property between fixture & removal body